Opiates+Amphetamines Test

(Ref. 311100)

DE	Produktinformation und Gebrauchsanleitung	2	CZ	Informace o produktu a návod k použití	16
EN	Product information and instructions for use	4	FI	Tuotetieto ja käyttöohje	18
FR	Informations sur le produit et instructions d'utilisation	6	SE	Produktinformation och bruksanvisning	20
ES	Información del producto e instrucciones de uso	8	DK	Produktinformation og brugervejledningen	22
IT	Informazioni ed istruzioni per l'uso	10	NL	Productinformatie en instructies voor gebruik	24
PL	Informacje o produkcie i Sposób użycia	12	NO	Produktinformasjon og bruksanvisning	26
PT	Informação sobre o produto e instruções de utilização	14		Symbols	28

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Instructions for the use: Opiates+Amphetamines Test

1. Intended use

As a preliminary test for detecting the presence of opium derivatives, especially heroin and morphine as well as drugs commonly used on the street drug scene, which contain these compounds, and amphetamines including ecstasy. The test can be used to assist police and customs officials working in the field of drug control.

2. Test Principle

Reagent suspended in a special adsorbent, available in 2 ml ampoules, which can be snapped open.

3. Sensitivity


The test has a sensitivity (lower detection level) of approx. 1 mg, depending on the purity of the sample. Doses of >5 mg should be avoided - small sample quantities will be more accurately detected.

4. Test procedure

- 4.1 Check whether any dark coloured grains are visible in the reagent of an ampoule. Such ampoules cannot be used.
- 4.2 Tap the unopened ampoule several times against a hard surface, such as a table top. By doing this, particles which may be stuck to the inside of the ampoule wall will fall to the bottom.

- 4.3 Carefully remove the polyethylene sealing. Ensure that the powdery reagent contained in the cap does not fall out.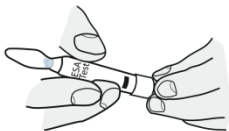


- 4.4 Open the ampoule as follows: Hold the ampoule firmly at the bottom with one


hand. Take the ampoule tip between the index finger and thumb of the other hand and snap it off at the predetermined breaking point by lateral pressure.

- 4.5 Transfer a small quantity of the sample into the ampoule, for example by using the enclosed spatula. Whilst transferring the sample, make sure to hold the ampoule in an upright position so that the substance falls to the bottom of the ampoule.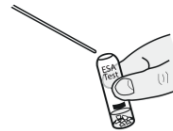

- 4.6 Allow the sample and the reagents to mix. This can be done by tapping with a finger or by repeated stirring with the spatula.
- 4.7 After completing the test, the ampoule must be sealed.

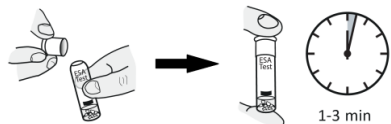5. Result Interpretation

All colour changes begin within just a few seconds. In all cases, **the colour will be fully developed after a minimum of one minute**. A further lapse of time produces a gradual deepening of the colour. **The evaluation of the test should therefore take place after no longer than three minutes:**

If opium or its derivatives (heroin, morphine) are present in the sample to be analysed, a change from colourless to **red/violet-blue** occurs.

In the presence of amphetamines, a **red-orange colour which gradually changes to brown** will develop.

If MDMA (ecstasy) is present in the sample being analysed, a colour from white to **dark purple/black** occurs.

Furthermore, the colour comparisons for selected substances can be consulted at the beginning of the package insert, as well as in the referenced literature.

6. Further Application Notes

- 6.1. To increase the reliability of detection, every ampoule contains an imprint with the colour changes.
- 6.2. Each ampoule can only be used once.
- 6.3. A separate spatula is provided for each test (10 spatulas in total).
- 6.4. Within the temperature range of -20°C - 55°C (-4°F - 131°F), no impairment of the ampoule's usability is to be expected.
- 6.5. Due to the gradual deepening of the colours, the ampoules cannot be retained for evidence.
- 6.6. The test is only suitable for the analysis of solids or powders, and not for liquids.

7. Specificity

Codeine will always show a blue-violet ^[1,2] colour, and fentanyl orange (darker depending on concentration). With amphetamine derivatives, as well as MDA, MDE and MDMA (ecstasy), a dark-blue/violet to black colour will develop ^[1,2]. Mescaline will be detected by a change from colourless to orange ^[2].

Furthermore, a variety of medicines might lead to similar or different colours. Every positive result must therefore be confirmed by a laboratory test.

Sugar does not affect detection. Within the recommended time it causes only a distinct light-yellow discolouration.

In order to provide the most versatile usage and to provide our customers with reliable results which take the above information into consideration, our information brochure 'Description of the indication behavior of ESA

Drug-Testers' lists roughly 220 individual substances together with their effect on indications. This list is frequently updated and the latest version may be obtained upon request.

8. Storage and Stability

If the ampoules are appropriately stored (protected from sunlight and temperatures above 25°C (77°F)), the expiry date indicated on the packaging applies.

9. Materials Supplied

The test kit provides materials for 10 tests and consists of:

- 10 snap-open ampoules
- 10 sealing caps
- 10 spatulas
- 1 set of instructions

10. Warnings and Precautions

10.1 The test has a very high level of operational safety – the risk of injury from burns or minor explosions (contains 25 µl concentrated sulfuric acid, H: 290-314, P: 280-301+330+331-305+351+338-308+310) is very low.

10.2 If, in spite of this, the test liquids come into contact with skin, eyes or clothes, then the affected areas should be immediately cleansed or rinsed with water as a precautionary measure.

10.3 The rapid test for opiates and amphetamines is designed for use in drug detection. No liability can be accepted in cases where the test is, or has been, inappropriately used.

10.4 Keep away from children.

11. Disposal

It is recommended to seal the used ampoules with the sealing caps, collect them in their original packages or in plastic containers and dispose of them in accordance with the current waste disposal regulations in the country of use.

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Rev. 1-02; 2017-11-06 KDy

Colour scale for selected substances (OPI/AMP and Meth/XTC Test)

Class/Substance	Colour (visual)	Colour (description)
	ca. 1. bis 5. Minute	Note colour deviations over time
Opiate/Opioide		
Buprenorphine		Violet ^[2]
Codeine		Light to dark blue-violet ^[1,2]
Dextromethorphan (DXM)		Dark violet to black
Desomorphine „Krokodil“		Violet to dark violet
Fentanyl (pure)		Orange to brown
Heroin		Red violet ^[1,2]
Methadone		Yellow-pink to orange ^[2]
Morphine		Light to dark violet ^[1,2]
Noscapine		Yellow
Opium		Red-violet (observe own colour) ^[1]
Oxycodone (Pill)		Pale violet ^[1]
Papaverine		Bright red-violet
Phenylethylamine		
2C-B		Yellow-green, green intensifies
2C-I		Yellow-green, dark green intensifies
25B-NBOMe		Yellow-green, intensifies (slow)
25I-NBOMe		Yellow-grey/green, green intensifies
Amphetamine		Orange to red-orange to red-brown ^[1]
Ethylone		Bright yellow
MDA (incl. Ecstasy)		Dark blue to black ^[2]
MDEA (incl. Ecstasy)		Dark violet to black
MDMA (incl. Ecstasy)		Dark violet to black ^[1,2]
MDPV		Bright yellow, positive for cocaine test
Mescaline		Yellow to brown ^[2]
Methamphetamine		Orange to red-orange to red-brown ^[1,2]
Methylone		Bright yellow
Other Substances		
Lidocaine		Light to dark red

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